

Heartsine AED Checklist

Client Name:

Date of Kit Audit:

Unit Identifier:

Person Conducting Check:



Check the Status Indicator

If the status indicator is not flashing or is flashing red a problem has been detected.

Check the Expiration Date of the Pad-Pack

Check the expiration date of the Pad-Pack currently inserted. If the Pad-Pak has exceeded its use by date, remove it from the unit replace with a new Pad-Pak. Ensure the Pad Pack is not opened or has visible damage.

Pad-Pack Expiry Date

Check Supplies

Check accessories and spares for damage or expiration. Replace any accessories found to be damaged or that have exceeded their expiration date.

Check Exterior of the Unit

Check the unit for cracks or other signs of damage.

Confirm Audio Prompts

Turn the unit on until an audio prompt is heard then turn off

If any of the checks identify issues, immediately contact APTS on 1300 325 001 or email store@auspactrianing.com.au